


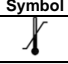

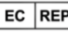




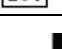

Streptococcus pyrogens	Typing strain T1 [NCIB 11841, SF 130]	1 x 10 ⁵ PFU/ml
Mycoplasma pneumoniae	Mutant 22	1 x 10 ⁵ PFU/ml
	FH strain of Eaton Agent [NCTC 10119]	1 x 10 ⁵ PFU/ml
	36M129-B7	1 x 10 ⁵ PFU/ml
Coronavirus	229E	1.5 x 10 ⁶ TCID ₅₀ /ml
	OC43	1.5 x 10 ⁶ TCID ₅₀ /ml
	NL63	1.5 x 10 ⁶ TCID ₅₀ /ml
	HKU1	1.5 x 10 ⁶ TCID ₅₀ /ml
Human etapneumovirus (hMPV) 3 Type B1	Peru2-2002	1.5 x 10 ⁶ TCID ₅₀ /ml
Human Metapneumovirus (hMPV) 16 Type A1	IA10-2003	1.5 x 10 ⁶ TCID ₅₀ /ml
Parainfluenza virus	Type 1	1.5 x 10 ⁶ TCID ₅₀ /ml
	Type 2	1.5 x 10 ⁶ TCID ₅₀ /ml
	Type 3	1.5 x 10 ⁶ TCID ₅₀ /ml
	Type 4A	1.5 x 10 ⁶ TCID ₅₀ /ml

Interfering Substances Reaction

When tested using the Novel Coronavirus (SARS-Cov-2) Antigen Rapid Test Cassette (swab), there was no interference between the device reagents and the Potential interference substances listed in below table that would create false positive or negative results for SARS-Cov-2 antigen.

Substance	Concentration	Substance	Concentration
Mucin	100µg/mL	Acetylsalicylic acid	3.0 mM
Whole Blood	5% (v/v)	Ibuprofen	2.5 mM
Biotin	100µg/mL	Mupirocin	10 mg/mL
Neo-Synephrine (Phenylephrine)	5%(v/v)	Tobramycin	10µg/mL
Afrin Nasal Spray (Oxymetazoline)	5%(v/v)	Erythromycin	50µM
Saline Nasal Spray	5%(v/v)	Ciprofloxacin	50µM
Homeopathic	5%(v/v)	Ceftriaxone	110mg/mL
Sodium Cromoglycate	10 mg/mL	Meropenem	3.7µg/mL
Olopatadine Hydrochloride	10 mg/mL	Tobramycin	100µg/mL
Zanamivir	5 mg/mL	Histamine Hydrochloride	100µg/mL
Oseltamivir	10 mg/mL	Peramivir	1mmol/mL
Artemether-lumefantrine	50µM	Flunisolide	100µg/mL
Doxycycline hyclate	50µM	Budesonide	0.64nmol/ L
Quinine	150µM	Fluticasone	0.3ng/mL
Lamivudine	1 mg/mL	Lopinavir	8µg/mL
Ribavirin	1 mg/mL	Ritonavir	8.2mg/mL
Daclatasvir	1 mg/mL	Abidor	417.8ng/mL
Acetaminophen	150µM	Pooled human nasal wash	N/A

SYMBOLS

Symbol	Meaning	Symbol	Meaning
	In vitro diagnostic medical device		Storage temperature limit
	Manufacturer		Authorized representative in the European Community
	Date of Manufacture		Use by date
	Do not reuse		Consult instruction for use
	Batch code		Meet the requirements of EC Directive 98/79/EC



HANGZHOU REALY TECH CO., LTD.
4th Floor, #12 Building, Eastern Medicine Town,
Xiasha Economic & Technology Development,
310018 Hangzhou, Zhejiang, P. R. China
Website: www.realytech.com



Luxus Lebenswelt GmbH
Kochstr.1,47877, Willich, Germany



Number:1101381601
Version:1.604
Effective Date:2020-10-29